


STRIPPABLE FIBER INSTRUCTIONS

Dear Picasso | Picasso Lite Customer,

Please follow these steps in order to properly connect the strippable fiber to your laser:



1. Disconnect the old fiber from the laser and un-wind it from the spool.

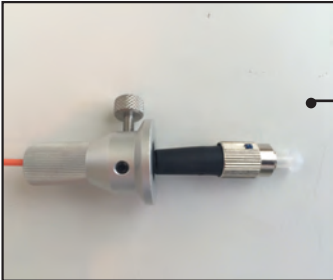


2. Remove the new fiber from the packaging and carefully wind it on the spool. Take care not to kink the optical fiber.



3. Loosen the adjustment set screw on the aluminum collar and slide the collar backwards to expose the connector.

4. Remove the protective dust cap and screw the fiber to fiber port on the back of the laser. Take care not to touch the fiber tip.

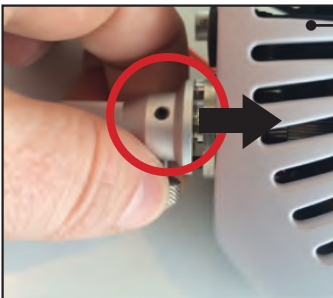

5. Slide the aluminum collar back over the connector so it is firmly pressed against the back of the laser as far as it can go and the large thumb screw is on the bottom as shown below. Ensure that the black plastic switch is being fully pressed by the aluminum collar as shown. While the collar is against the back of the laser, tighten the thumb screw to secure the aluminum collar on the laser.



8925 N. Meridian St. Suite 250
Indianapolis, IN 46260 USA

www.amdlasers.com
866.999.2635